



INSTRUCTIONS FOR USE
INDICATIONS,
CONTRAINDICATIONS,
WARNINGS.

MicroSilver Spray

WOUND CARE DRY SPRAY

PRODUCT DESCRIPTION:

MicroSilver Spray is a topical spray powder dressing containing hyaluronic acid sodium salt (HA) and metallic silver. Hyaluronic acid binds a large amount of water and forms a viscous hydrated gel even at low concentrations. During the wound repair process HA allows the rapid diffusion of water-soluble molecules and migration of cells. MicroSilver BG™ inhibits microbial contamination and keeps the wound environment free from additional exogenous bacterial sources. HA and silver are joined to a matrix (light kaolin and silicon dioxide) with absorbent properties. This particular formulation of MicroSilver Spray controls the exudates creating an environment that facilitates the natural healing process and provides an effective barrier to microbial penetration. The product remains in site and is unaffected by stretching from wound borders. It can be easily removed by cleansing the lesion with physiological solution.

COMPOSITION:

Principal component: Hyaluronic acid sodium salt at 0.2%

Other components: Metallic Silver 2.0%, Silicon Dioxide, Light Kaolin and n-Butane as propellant.

INDICATIONS:

MicroSilver Spray is indicated for a temporary local management of non-infected cutaneous lesions (i.e. abrasions, excoriations, fissured skin, cuts, minor surgical wounds, abscesses, sores, localized first and second degree burns) and to provide a moist environment protected against exogenous additional bacterial sources.

CONTRAINDICATIONS:

Patients with ascertained individual hypersensitivity to the product components must not use MicroSilver Spray.

PRECAUTIONS AND WARNINGS:

- It is suggested not to use for a period longer than four weeks.
- Consult your veterinarian in case of incomplete healing process or infection complication after a short period of time.
- Protect the patient's eyes, nose, or mouth when spraying on the higher part of the body.
- Do not spray near or onto the eyes, nose, or mouth.
- Avoid breathing the powder.
- In case of accidental ingestion or inhalation consult a veterinarian or physician immediately.
- Avoid direct contact of the container with the wound.
- Do not use the product concomitantly with proteolytic enzymes or disinfectants containing quaternary ammonium salts.
- After each application put the cap back on the container.
- Do not use in case of package damage.
- The expiry date refers to the product stored correctly.
- Content under pressure and flammable. Do not puncture or incinerate container. Do not use or store near any heat source or open flame. Do not store in direct sunlight.
- Exposure to temperatures above 50°C may cause container to explode.
- After use, dispose according to applicable national regulations.
- Keep out of reach of children and animals.

UNDESIRABLE EFFECTS:

The following local adverse reactions may occasionally occur: application site rash, itching, burning sensation, skin irritation, skin discoloration. The symptoms are usually transient, of mild severity and rarely require early termination of treatment.

INSTRUCTION FOR USE:

The lesions should be cleansed prior to treatment. Shake the container well before use. Holding the container in vertical position, spray generously to cover the wound surface.

Spray at intervals for short periods of time. After the application of MicroSilver Spray, it is suggested to cover the medicated area with a sterile bandage.

Change the dressing every 2-4 days as required. Reapply the spray with each changing of the dressing.

ATTENTION: THE NOISE DURING APPLICATION CAN ALARM THE PATIENTS, IT IS THEREFORE, RECOMMEND TO HOLD THEM IN PLACE AND COVER THEIR EARS DURING APPLICATION.

Apply regularly to the freshly healed skin once or twice per day if possible. Consult with your veterinarian to assess healing.

PRESENTATION:

75 ml pressurized container with 10 g of spray powder.

HANDLING AND STORAGE:

Handle with care and store in a dark and dry place at a temperature below 40° C.

CAUTION: FOR ANIMAL USE ONLY. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED VETERINARIAN.

FOR TOPICAL USE ONLY.

AVOID DIRECT CONTACT WITH EYES. IN CASE OF CONTACT, FLUSH EYES WITH WATER AND SEEK MEDICAL ATTENTION IF IRRITATION PERSISTS.

KEEP OUT OF REACH OF CHILDREN AND ANIMALS.

NOT FOR USE IN HUMANS.

SEE INSTRUCTIONS FOR USE AND APPLICATION.

SYMBOLS

	COMPRESSED CONTENTS
	OXIDIZING AGENT
	CONTAINER MAX VOLUME
	ADD COMPLIANT
	USE WITHIN 6 MONTHS OF OPENING
	ESTIMATED CONTENTS

MANUFACTURED BY:

Bio-Gate AG
Neumeyerstrasse 28-34
90411 Nuremberg
+49 (0) 911/47 75 23-100
www.bio-gate.de

DISTRIBUTED BY:

BioMedtrix, LLC
9 Whippany Road, Suite B2-7,
Whippany, NJ 07981
973-331-7800
www.biomedtrix.com

Veterinary Orthopedic Implants
310 Commerce Lake Drive,
Suite 107,
St. Augustine, FL 32095
904-436-6540
www.vetimplants.com